

### ERBE VIO dV® 2.0 Generator with Power Limit Settings

Control energy output in fine increments using a combination of Mode, Effect, and Power Limit settings with the new ERBE VIO dV 2.0 software. Use the table below to compare watt-based energy settings to similar initial output settings for the ERBE VIO dV 2.0 generator with Power Limit settings. The table is an approximation of constant power ESU output. Additional changes may be necessary to achieve desired tissue effect, please refer to ERBE VIO dV user manual for guidance.

**MONOPOLAR  
COMPARISON**



From		To			
Watt-based energy settings		VIO dV 2.0 energy settings			
Mode	Power Setting (watts)	Mode	Effect	Power Limit	Foot Pedal
Fulgurate	20	Swift	2	20	
Fulgurate	30	Swift	3	30	COAG
Fulgurate	40	Swift	4	40	
Fulgurate	50	Swift	5	50	
Pure Cut	25	Auto	2	25	
Pure Cut	35	Auto	3	35	CUT
Pure Cut	45	Auto	4	45	
Pure Cut	55	Auto	5	55	
Coagulation	20	Soft*	3	20	
Coagulation	30	Soft*	4	30	COAG
Coagulation	40	Soft*	5	40	
Coagulation	50	Soft*	6	50	

\*Soft Mode on bipolar can be used with or without the Autostop feature.

**BIPOLAR  
COMPARISON**



**Important Safety Information**

Be sure to read and understand all information in the applicable user manuals, including full cautions and warnings, before using da Vinci products. Failure to properly follow all instructions may lead to injury and result in improper functioning of the device. Training provided by Intuitive Surgical® is limited to the use of its products and does not replace the necessary medical training and experience required to perform surgery. Procedure descriptions are developed with, reviewed and approved by independent surgeons. Other surgical techniques may be documented in publications available at the National Library of Medicine. For Important Safety Information, indications for use, risks, full cautions and warnings, please also refer to [www.davincisurgery.com/safety](http://www.davincisurgery.com/safety) and [www.intuitivesurgical.com/safety](http://www.intuitivesurgical.com/safety). Unless otherwise noted, products featured are available for commercial distribution in the U.S. For availability outside the U.S., please check with your local representative or distributor.

© 2018 Intuitive Surgical, Inc. All rights reserved. Product names are trademarks or registered trademarks of their respective holders.