

# Software Update: Key Changes

## For da Vinci Xi and da Vinci X Surgical Systems

Software Version: da Vinci OS4 v11 (P11)

Representative changes in this update are described below. For more details, refer to the user manual.


### Intuitive Account Login Updates

Updates have been made to the Intuitive account login process. Surgeons will now have the ability to:

- Designate a primary procedure in a multi-procedure case.
- Designate a primary surgeon for the primary procedure in a multi-surgeon case.
- Manage their procedures while another surgeon is logged in to the same case.
- Change surgeons during a multi-surgeon case.

#### Changing the primary procedure:

- The surgeon can now add multiple procedures to the case, and then select the primary procedure for the case.
- Access this screen from the OVERVIEW tab, then select CASE OVERVIEW, and tap SET AS PRIMARY on the desired procedure.
- Tap DONE to proceed to the instrument tab.

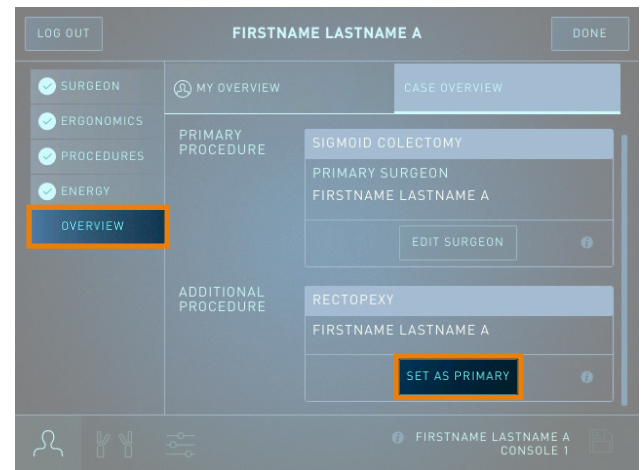

#### Changing the primary surgeon:

- The primary surgeon will complete the login process and then log out, allowing an additional surgeon to complete the login process.
- Upon finishing their login process, the additional surgeon can select EDIT SURGEON and select their name from the list provided.
- Access this screen from the OVERVIEW tab, the CASE OVERVIEW tab, and tap EDIT SURGEON.
- Tap DONE to return to the OVERVIEW tab.



## Intuitive Account Login Updates Continued


### Managing procedures with two surgeons logged in:

- When you log in after another surgeon logs in during the same case, you can change, add, or remove procedures under your account.
- From the PROCEDURES tab, select or unselect any listed procedures by tapping the procedure name.
- Tap NEXT to return to the OVERVIEW tab.



### Changing surgeons during a case:

- While only one surgeon can be logged in to a console at a time, there is no limit to the number of surgeons that can log in during a case.
- Access this screen from the SURGEON tab and then tap the designated surgeon account from the list of CURRENT CASE SURGEONS.
- Tap NEXT to return to the OVERVIEW tab.

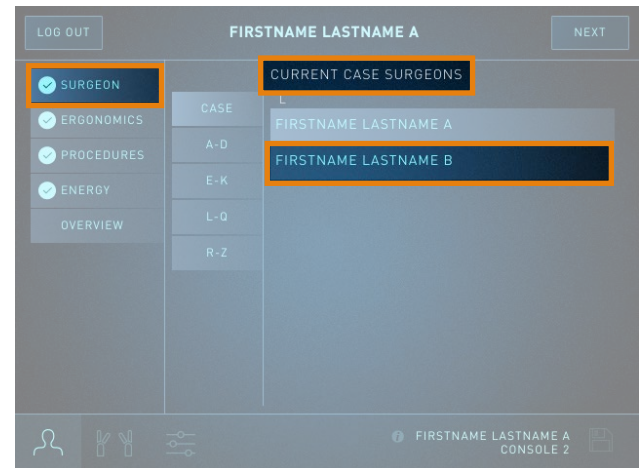

## TilePro Sizing Updates

Updates have been made to the TilePro sizing controls. Surgeons will now have the ability to switch between three pre-defined TilePro sizes (small, medium, large).

- From the SETTINGS tab, tap the TILEPRO button and select from the three TILEPRO SIZE options.

**i Note:** In dual console mode, the console must be assigned all active instruments in order to make TilePro size changes. If a console that is not assigned all active instruments attempts to make a TilePro size change, a system message appears.



---

**Disclosures**

Intuitive technology should only be used by physicians and staff who have received specific training in the use of Intuitive technology. Training provided by Intuitive is limited to the use of Intuitive technology, instruments, and accessories. It does not replace the necessary medical training and experience required to perform surgical procedures. Nothing in Intuitive's training curricula makes expressed or implied promises or commitments to the persons attending. This includes NO promise or commitment that the trainee will be provided any specific product or technology in the future or that the trainee will be certified to perform a medical procedure related to the curriculum. Some of the training described in Intuitive promotions may be provided by organizations or entities other than Intuitive.

**Important Safety Information**

Europe: Medical devices, (Intuitive) CE 2460, (Erbe) CE 0124, refer to Instructions For Use for further information.

For intended use and/or indications for use, risks, cautions, and warnings and full prescribing information, refer to the associated user manual(s) or visit <https://manuals.intuitivesurgical.com/market>.

**Product Availability**

Some products, features or technologies may not be available in all countries. Please contact your local Intuitive representative for product availability in your region. Refer to the product specific user manual for indications, contraindications, warnings and other product information.

**Privacy Notice**

Intuitive's Privacy Notice is available at [www.intuitive.com/privacy](http://www.intuitive.com/privacy).

© 2023-2024 Intuitive Surgical Operations, Inc. All rights reserved. Product and brand names/logos are trademarks or registered trademarks of Intuitive Surgical or their respective owner.